halo

microchip scanner

User Manual

EN: User Manual	1-5
FR: Guide d'utilisation	6-10
DE: Bedienungsanleitung	11-15
NL: Handleiding	16-20
ES: Manuale de usuario	21-25
IT: Manuale d'uso	26-30

www.haloscaner.com


halo microchip scanner

Product Diagram


What's in the box?

Check that you have all these parts.

- 1) Halo Scanner
- 2) USB Cable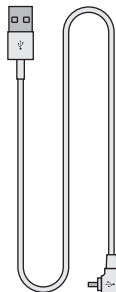

How to take a reading

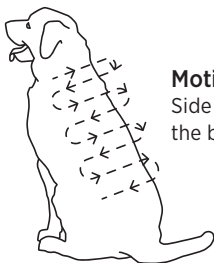Motion Pattern 1
Side to Side across
the body

Motion Pattern 2
Up and down
the body

Halo Microchip Scanner

The **Halo Microchip Scanner** complies with ISO 11784 and ISO 11785 norms. The **Halo** is Scanner Angel® compatible and is supplied with a USB lead for charging the **Halo's** internal rechargeable battery.

Recharging

Before using your **Halo** for the first time, it will need to be charged for 3½ hours.

will appear on the display when the battery is running low with the message “Battery Low”. To recharge the battery, plug the USB cable into the back of the **Halo** and into an available USB port on either a computer or wall plugin charger. During the charging cycle, the display will switch off. To see if the battery has finished charging, press the start button and the display screen will inform you if it is “charging” or “charged”.

will appear on the display when the battery is fully charged.

Available Software

Depending on your application, there is a range of software available including the Scanner Angel® software for the **Halo** which can be downloaded from www.haloscanner.com. Please note there is no Mac OS compatible software at the moment.

You do not need to install any software to use the **Halo** as a normal traditional microchip scanner.

Scanning

To start your **Halo Scanner**, simply push the on/off  symbol on the back panel and release. The display will show “Halo Scanner Scanning” and the scanner will emit a single beep. If no valid microchip is found within 20 seconds, then the display will show “No Tag Found” and the scanner will again emit a single beep. To start scanning again just press the  symbol again. The **Halo** will automatically switch off after 50 seconds.

The **Halo** will also display the microchip's temperature of compatible temperature sensing microchips. The temperature will only be displayed if the temperature is between 25°C and 43°C.

The outer surface of your **Halo** is the scanning area and when a valid microchip is found the scanner will emit a double beep and the microchip number will be shown on the display.

To review previously scanned numbers, wait until the unit has powered down. Then press and hold the power button on the back for four seconds and then release. To scroll through the list, press the power button once to see the next number. To exit this mode, press and hold the power button for four seconds and release.

Cleaning

Your **Halo** should never be immersed in liquid or placed in any sterilization equipment. It can be wiped with an antibacterial wipe.

Disposal of products and used batteries

This symbol on the product and / or accompanying documents means that used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated recycling points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product. Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling.

Warranty

The **Halo Scanner** comes with a 2 year return to base warranty from the date of purchase. The warranty is limited to any defects in materials components or manufacture including any defects in workmanship under normal use. Faults arising from misuse, neglect, disassembly are not covered by this warranty.

If a fault arises which is covered by the warranty then the manufacturer will either replace or repair the product free of charge at the manufacturer's discretion. In the event of requiring warranty support then the manufacturer should be contacted in the first instance on info@idporte.com. Your statutory rights are not affected.

Disclaimer

The **Halo Scanner** and associated facilities is not intended to replace traditional Veterinary advice. Microchip temperatures are not sufficient for determining the status of an animal's health and should be determined by a Veterinary professional. We expressly disclaim any liability for any loss, damage or fees that arise from using this product.

Technical Specifications

- In compliance with ROHS, CE and ISO 11784 norms.
- Dimensions: 135mm diameter x 33mm
- Weight: 163g
- Boxed Dimensions: 147mm x 144mm x 38mm
- Boxed Weight: 250g
- Scans per Charge: 8 second scans x 1000
- Standby Time: 2 months (typically)
- Battery: Internal lithium polymer rechargeable
- Recharge via USB port
- Charge Time: 3.5 Hours
- Reads: FDX-B, FDX-A (FECAVA), EM4102
- Reads the temperature from Thermo chips
- Operating Temperature: 0°C to 50°C
- Display: LCD 2 lines x 16 characters
- Tones: Audible beep
- Memory: Last 2000 scanned numbers
- Operation: Simple to start scanning and automatic turn off after 50 seconds
- Communications: USB port
- Manufactured in Great Britain
- Colour Options: White, Black, Pink, Blue, Green, Purple

All specifications subject to change without prior notice.

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